



HCP Screening Audiometers from Navy MTFs

Types of audiometers:

Benson Medical CCA-200 and CCA-200mini

Process for the annual calibration of HCP screening audiometers:

- An Electronics Technician, from NMCPHC's Calibration Laboratory, will contact you prior to your audiometers coming due for calibration (Navy MTF HCP customers).
- Loaner audiometers assessment will occur and a timeframe your audiometer calibration will establish.
- Loaner audiometers will be shipped, if needed.
- Send your audiometers to the Calibration Laboratory at NMCPHC during the timeframe agreed upon.
 - Ensure power to the audiometer is off before disconnecting it (and later reconnecting it after calibration).
 - Use the pelican case that contained the loaner audiometers to ship your audiometers to the Calibration Laboratory. A FedEx shipping requisition is provided to pay for shipping to NMCPHC.
 - Include a completed form DD1149 in the shipping case. Ensure that your command's name, mailing address, point of contact, telephone number, and email address are complete and accurate.

Navy MTF HCP customers should never send an audiometer in clinical use to NMCPHC without having a loaner from NMCPHC or one of their own spare audiometers to use during the calibration period.
- Upon completion of calibration, your audiometers will be shipped back to you via FedEx.
- Return the loaner audiometers in the pelican case provided.
- Contact the Calibration Laboratory if you have any questions.